

## DQE

## Heavy Duty Privacy Screen MC5032

## **Disassembly & Storage**

- A. After use, the screen and frame should be fully decontaminated using your standard operating procedures and in full compliance with state and federal regulations. Allow product to completely dry.
- B. Disconnect each Side Pole (#4) from Lower Cross Member (#2).
- C. Place Screen Assembly (#3) flat on the ground and disconnect each Side Pole (#4) from Upper Cross Member (#5).
- D. Swing the bottom end of one Side Pole up to the opposite end of the Upper Cross Member.
- E. Repeat last step with other Side Pole. The screen will be in a triangle shape.
- F. Disconnect Legs from Lower Cross Member. Lay Lower Cross Member and Legs beside Side Poles.
- G. By rolling together, wrap all frame pieces in screen material.
- H. Insert bundled screen roll with instruction sheet into Canvas Carry Bag.